SHERPA

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Reliable bearers

SHERPA®

Montascale a cingoli - *Tracked stair climber*

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...reliability from a far-away land... The name chosen for this line of stair climbers comes from the mountains of Nepal, the home of the Sherpa, a people whose culture is founded on respect for human dignity. Their great resistance to fatigue and sociable character, together with the congenital ability to adapt to great heights, has made them the perfect "high-altitude bearers" and essential companions for the first mountain-climbing expeditions, capable of carrying the heaviest loads and making trails safe so that climbers could reach the peaks... and these characteristics are the ones that distinguish SHERPA stair climbers, whose goal is to become a faithful and reliable travelling companion.

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N 909 allows carrying a lot of types of wheelchairs on stair even with very low frames.

- Release and safety controls of the tiller are placed on the handlebar of the tiller itself.
- Load Kg. 150.
- Reinforced tiller thickness.
- Ergonomic handle for a practical use.
- Adjustable fixing clamps on two axles.
- Support ramps in painted sheet-steel coated in non-slip aluminium.

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In addition to the N 909 features, this version is equipped with 2 additional motorized wheels that can be activated during the landing manoeuvring.

SHERPA N 909/TM

Montascale a cingoli con rampe. Versione Full-optional
Tracked stair climber with ramps. Full-optional version

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In addition to the N 909 features, this version is equipped with:

- 2 additional motorized wheels that can be activated during the landing manoeuvring.
- Powered tilting system of the tiller.

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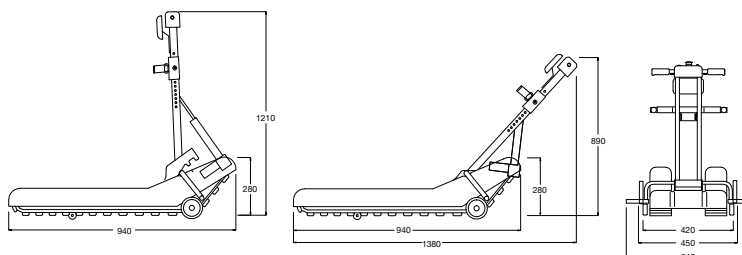
- Load 130 kg.
- The powered tilting system of the tiller avoids any efforts by the operator, it considerably improves the comfort and it significantly reduces the manoeuvre area.
- The 2 motorized castors, actionable when maneuvering directly from the tiller, make Sherpa practical and manageable to drive.
- The new folding supportive arms for wheelchair, reduce a lot the overall dimensions during storage and transport.
- The release and safety controls of the tiller, placed on the handlebar of the tiller itself, make it more easy and fluid to use.

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*The top of the new Sherpa line, full-optional usability,
fall down the limits of manoeuvre.*

Sistema Ruote motorizzate in azione (Wheel system).
Powered wheel system in action (Wheel system).

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- Load 130 Kg.
- The release and safety controls of the tiller, placed on the handlebar of the tiller itself, make easier and fluid the coupling of the wheelchair.
- The Folding supportive arms reduce the overall dimensions during storage and transport.

SHERPA N 906

Montascale a cingoli
Tracked stair climber

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- Load 150 kg.
- Same features as the model N 905 on a reinforced structure to ensure a greater payload.

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- Load 130 kg.
- In addition to the N 905 features, Sherpa N 907 is equipped with 2 motorized wheels that can be activated during the landing manoeuvring, directly from the tiller granting agility to the device.

 **WheelSystem**

Sistema Ruote motorizzate in azione (Wheel system).
Powered wheel system in action (Wheel system).

SHERPA

SHERPA N 902

Montascale a cingoli
Tracked stair climber

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- Load 130 Kg.
- Release and safety controls of the tiller are placed on the handlebar of the tiller itself.
- Folding supportive arms for wheelchair
- External battery charger (standard), external one on demand.

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Sherpa tracked stair climbers are indicated for disabled persons who need to navigate stairs in buildings without suitable equipment; for this reason, they are suitable for overcoming architectural barriers. The stair climber consists of a metal structure that moves on rubber tracks and a tiller, which allows steering and anchoring a wide range of wheelchairs. The control and emergency buttons are ergonomically arranged on the tiller. Movement is provided by an electric motor managed by an electronic controller for soft starts and stops along with a high level of comfort. The unit is battery powered. The battery-charger is built into the machine and the residual charge is displayed on the control panel. The track provides forward and backward movement while changes in direction are made by a pair of manoeuvring wheels that extend automatically. The machine is easy to transport because unhooking the steering column reduces its overall dimensions and weight, making it easy to manoeuvre. The body of the machine is activated by a button located on the rear control panel. The structure's loading-bearing frame is made of electrically welded steel treated with a cathoresis process, while the steering mechanism consists of a chrome-plated steel framework. The thermoformed high-impact polystyrene chassis gives the product a refined, modern look. The machine's extremely modest height makes it simple to attach a wide variety of wheelchairs.

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- Powered by irreversible gear motor and electronic control with soft start and stop operation.
- Automatic self-braking system in case of emergency or power failure.
- Maximum gradient: 70% equal to 35°.
- Power input: 24 VD, electronically controlled.
- Silent movement: elimination of "clamp" noises at start-up and stopping (no electromagnetic switches).
- Driving comfort: soft starts and stops, no vibrations.
- Safety: electronic and mechanical features ensure a high level of safety.
- Maintenance: the parts subject to wear are easy to replace thanks to easy access and the electronic connection system.
- Design: modern and functional.
- Quality: the control standards applied to the production process ensure a top quality product.

Special safety features include:

- Motor with "irreversible" mechanical transmission.
- Moving parts are thoroughly protected to prevent debris from entering the device.
- Power cord check with device to inhibit operation.
- Tiller attachment system complete with safety lock.
- Reduction of electromagnetic emissions.
- Emergency button for total inhibition of functions.
- Manual device for the movement of the machine in case of breakdown (except N 906/N909/N 909R/N 909TM).

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Technology in motion. The evolution continues

The logo for Yack, featuring the word "Yack" in a stylized, bold, blue font with a registered trademark symbol.

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...along the steep and arduous Himalayan trails, the Yack is the only animal capable of carrying heavy weights and people above 2000 meters. It is an indispensable companion for the inhabitants of those areas who rely on it when the trails are really difficult. Just like the animal it is named after, the Yack stair climber's mission is to overcome obstacles, and it has the ambitious goal of loading and transporting a greater number of wheelchairs in order to make everyday routes accessible, practicable and easy.

Yack wheeled stair climbers are indicated for disabled persons who need to navigate stairs in buildings without suitable equipment; for this reason, they are suitable for overcoming architectural barriers. The stair climber consists of a metal structure that moves on stairs using a translating strut system and a pair of wheels for moving on flat surfaces. It has a tiller that allows steering and anchoring a wide range of wheelchairs. The control buttons are ergonomically arranged on the tiller. It is driven by an electric motor with electronic controller that constantly monitors the functioning of the machine, ensuring optimum management of its performance. It is powered by a rechargeable battery that can be extracted and, thus, replaced; its remaining charge is displayed on the control panel. The machine is easy to transport because unhooking the tiller reduces its overall dimensions and weight, making it easy to manoeuvre. The machine body's load-bearing frame is made of high-gauge anodized aluminium, while the steering column is made of painted, double structural steel. The covers are painted steel.

YACK

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Tough for any obstacle.

YACK N 910
Montascale a ruote
(macchina base)

Wheeled stair climber (main machine)

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N 911 configuration is designed to transport persons who are directly seated on-board the machine. It has a small padded seat with foldable armrests and footrest and height-adjustable headrest. The N 911's reduced size makes it easy to manoeuvre in narrow spaces. The option of disassembling it into several parts makes the machine extremely easy to transport.

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Wheeled stair climber with platforms for wheelchair coupling

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This configuration which has width-adjustable lower plates anchored to the body of the machine and width- and height-adjustable upper hooks anchored to the body of the steering column, allows the rapid positioning and anchoring of most wheelchairs on the market today. The versatile connection plates of the N 912 make it a nearly universal stair climber, allowing to accommodate and safely transport a wide range of wheelchairs. Practical and very simple to use, it is easily transported thanks to the option of disassembling it into two parts.

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This configuration has a lower bar and sliding upper hooks. This solution, together with a sliding bracket system applied to the wheelchair, allows it to be quickly positioned and anchored on-board the machine. This solution allows using the wheelchair normally on flat surfaces while keeping the stair climber attached at the same time. When it becomes necessary to deal with stairs, a simple manoeuvre removes the wheels of the chair, reducing its overall dimensions and, thus, facilitating drivability. The N 913 is the most versatile of the Yack line because it is possible to configure it in the two alternative N 911 and N 912 versions with a series of accessories.

In the N 913 configuration, Yack becomes a stair climber is easy to use. It has an extremely simple method of hooking to most types of wheelchairs and carries them on very narrow stairs with reduced manoeuvring space because it allows detaching the wheels of the chair and reducing its overall dimensions to a minimum. Plus, the option of disassembling the machine into two parts makes it easy to transport.

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Once hooked on, the stair climber becomes a single compact unit and an integral part of the wheelchair itself. By keeping it slightly lifted off the ground, N 913 moves together with the wheelchair and is always ready to face any obstacle that arises along your route.

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*All versions of the machine use the same body
and the easy-to-apply accessories make
transformations simple...*

*So, it would be easy to detach the N 913's sliding arm,
replace it with the backrest and position the seat
to transform it into an N 911. Similar manoeuvres
allow attaching the sliding arm and wheelchair
connection plates to turn it into an N 912. This
important feature makes this stair climber capable
of adapting to the changing needs of its users.
It is a true example of "technology in motion" that
is constantly "evolving" in an attempt to make it
increasingly easier to overcome every barrier.*

YACK

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Montascale a ruote per carrozzina
Wheeled stair climber for wheelchairs

YACK

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Montascale a ruote agganciabile alla carrozzina
Wheeled stair climber attachable to the wheelchair

YACK

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Montascale a ruote con poltroncina
Wheeled stair climber with integrated seat

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Equipped with dual operating controls to ensure easier operation for the carrier.

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On/off jack. When the device is not in use for a prolonged period of time the sleep-mode (stand by) function is activated, followed by an audible warning. The rapid removal of the jack acts as an emergency system that brings the moving machine to an immediate halt.

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External electronic battery charger. The battery can be charged either inside or outside the device, allowing it to be powered with a supplementary battery pack.

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The device's on-board electronics control its functioning and prevent operation if the conditions of use are not optimal for safety, such as if the residual charge is insufficient or the on-board load is higher than the maximum allowed. A well-designed operator panel makes it extremely easy to set one of the 4 available speeds, select between continuous and single-step operating mode and monitor the residual battery charge.

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Adjustable handle that allows the housing in tight spaces.

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Adjustable steering mechanism or "tiller" that makes the device easy to manoeuvre on stairs. The YACK tiller is fitted with an ergonomic handle that makes it even easier to control.

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Automatic stair-end brakes. This mechanical safety system allows YACK to automatically brake at the edge of the step.

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Ergonomic handwheel for tiller lock.

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Width-adjustable telescoping hooking bar with handwheel adjustments.

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Upper hooks with floating system automatically adjustable according to the inclination of the wheelchair to ensure maximum stability.

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YACK was created to move up and down any kind of staircase, even on very narrow stairs, steps with protruding edges, and on winding or spiral staircases. Since it requires limited manoeuvring space, it can also be used in small areas. YACK's intricate design and advanced electronic technology make it an extremely refined, reliable and safe product. An understated and elegant design, paired with extreme manoeuvrability, make this a truly cutting-edge product.

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YACK's operating phases, step by step. Single step or continuous operation.