CE

Confortaid

 by  KATEDRA
mod. brevettato

 PRATICO
EFFICACE

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Anti bedsores water cushion

EFFECTIVE
HANDY


CONFORTAID is a cushion made of PVC so as to be filled with water.

CONFORTAID, thanks to physical properties of the water (Pascal's principle),:

- **Preserves** the blood circulation for weak patients' ischemic tissues.
- **Prevents** the bedsores from forming or helps their treatment.
- **Contributes** to shorten stays in hospital, allowing for completing the treatment at patient's home; in fact CONFORTAID is handy and inexpensive.

To protect the sacral region and the coxofemoral joints, two Confortaids must be placed correctly on the bed (fig. 1); only one cushion is needed for each heel (fig. 2) or elbow: if the shoulders also need protection, a couple more cushions may be placed on the bed (fig. 3). CONFORTAID may be used as a pillow (fig. 4a and 4b).

CONFORTAID must be filled only with water.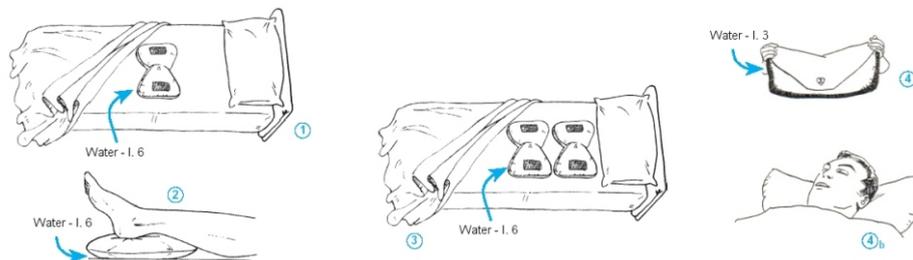

INSTRUCTIONS FOR USE

- Fill half cushion equally with warm water in winter.
- **Expel the air carefully** and close the valve.
- Set two cushions - equally filled - on the bed (fig. 1) so the points and the marks overlap by 10-14 cm, depending upon the patient's body size: insure that the patient's buttocks lie on the Confortaids' dotted zone (fig. 2).

Confortaid is washable with mild soap and warm water.

Dispose of Confortaid properly to protect the environment.

Handle with care: the manufacturer is not responsible for tearing due to accidents or carelessness.