



L.go Marzabotto, 27
37126 Verona - Italy
tel. 39 045 8302932
e-mail: perali.l@tin.it

CONFORTAID

[Products](#)

[Publications](#)

[Curriculum](#)

Chairs

CONFORTELLA

SUPERNOVA

EURA

Anti bedsores water cushion

CONFORTAID

WHEELCHAIR
CONFORTAID




CONFORTAID is a cushion made of PVC so as to be filled with water.

CONFORTAID, thanks to physical properties of the water (Pascal's principle):
Preserves the blood circulation for weak patients' ischemic tissues.
Prevents the bedsores from forming or helps their treatment.
Contributes to shorten stays in hospital, allowing for completing the treatment at patient's home: in fact CONFORTAID is handy and inexpensive.

To protect the sacral region and the coxofemoral joints, two Confortaids must be placed correctly on the bed (fig. 1); only one cushion is needed for each heel (fig. 2) or elbow: if the shoulders also need protection, a couple more cushions may be placed on the bed (fig.3).

CONFORTAID may be used as a pillow (fig. 4a and 4b).

CONFORTAID must be filled only with water.

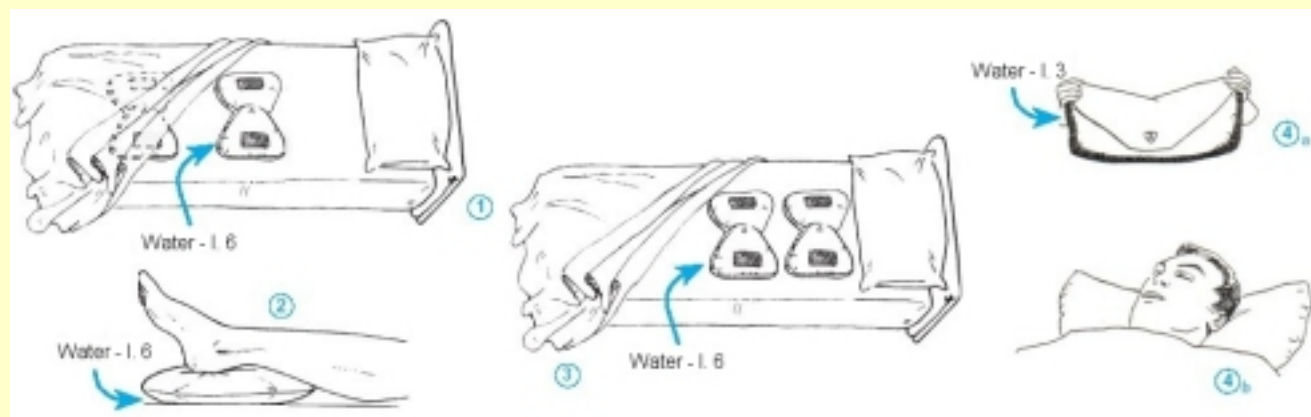

e-mail: perali.l@tin.it