 **Airgo**[®]

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Table of contents

Preface	11
Declaration of conformity	11
Service life	11
Disposal	11
Identification plate	11
Reuse	11
Warranty	11
Parts of the product	12
Intended purpose	13
Indication	13
Scope of delivery	13
Safety instructions	13
Fitting the footrests	13
Unfolding	14
Assembly of accessories	14
Height adjustment of push handles	14
Adjusting the brakes	14
Braking Parking	14
Use as a rollator	15
Use as a transport chair	15
Folding Transporting Stowing	15
Cleaning Maintenance	15
Technical data	16

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.


This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

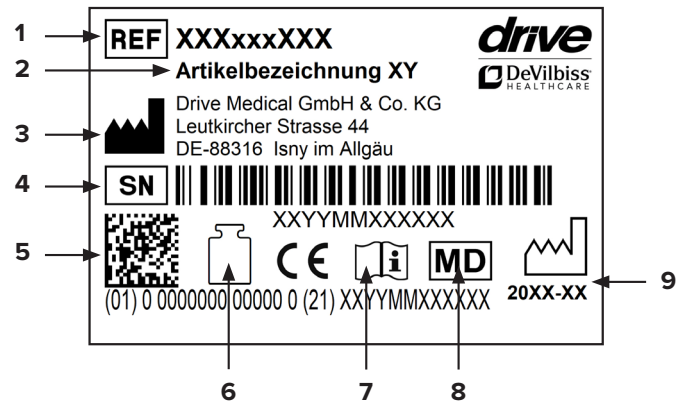1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

EXCLUSION OF WARRANTY

Parts subject to wear include (among others): wheels, brakes and handles

Parts of the product

Airgo Fusion

- | | | |
|-----------------------------------|---------------------------|------------------|
| 1. Handle | 6. Rear wheel | 11. Bag |
| 2. Push handle | 7. Front wheel | 12. Footrests |
| 3. Brake handle | 8. Brake | 13. Bowden cable |
| 4. Locking screw for push handles | 9. Brake adjustment screw | 14. Folding lock |
| 5. Seat | 10. Backrest | |

Intended purpose

Walkers are used to support safe walking and, depending on the design, can also be used to relieve pressure. Four-wheel walking frames are products made from tubular frames which have four solid rubber wheels. These walking frames are designed such that the user always moves between the contact points or wheels. These walking frames have braking systems which operate using Bowden cables operated by handgrips. These walking frames can be folded for transportation and the handgrips are height-adjustable to suit different heights.

Indication

INDICATION:

Walking aid for outdoor and indoor use to compensate for or improve limited walking ability. Insured parties are to be provided with walking aids with the intention of compensating for or at least improving the user's limited walking ability (expansion of radius of activity or mobility). Persons with limitations of hand/arm functions require particular adjustments/designs of handles or handrests/armrests. Movable walking aids may not be suitable for persons with balance disorders; in individual cases, this should be tested out. Where appropriate, an attendant must be present. The rollator is to be used only for normal, everyday activities.

CONTRAINDICATION:

The rollator is not suitable for use in the case of: perception disorders, severe loss of balance, inability to sit down.

Scope of delivery

1 x	Rollator
1 x	Pocket
1 x	Back support
2 x	Footrests
1 x	Instructions for use


Safety instructions

- Use the rollator only as a walking aid!
- Use the rollator only on an even and solid surface.
- Rollator may not be used as a seat in vehicles, such as a bus or train!
- It is prohibited to stand on the rollator or to transport a person or animal on the rollator.
- Take note of the permitted maximum weight of the user.
- This information can be found in the technical data in these instructions for use as well as on the identification plate.

- Use the tray only for small, lightweight objects which are not sharp or hot. Max. load: 5.0 kg
- Use the basket only to transport objects and do not overload it! Max. load: 5.0 kg
- Use on escalators and the like is expressly prohibited!
- Do not hang any bags on the handle – risk of tipping!
- On steep surfaces, use the service brake (pull the brake lever upwards).
- Leaning excessively on the handles can cause the rollator to tip backwards.
- When folding/unfolding the rollator, clothing and body parts can be pinched.
- Protect the rollator from direct sunlight. Product parts can become hot and cause skin injuries if touched.
- Prior to each use, ensure that your rollator is intact and all adjustable parts are properly secured. Check that the brakes are working properly!
- Before using the rollator, ensure that the folding guard is properly locked.
- Check the screws on the rollator regularly and tighten, if necessary.
- The braking effect can be impaired by wear and tear on the tires. For this reason, check the braking effect and tires regularly.
- The rollator should NOT be carried by the seat, removable parts or the brake cables.
- Every serious incident which occurs in connection with the product must be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is located.

Fitting the footrests

Match the letters on the frame (L and R) to the same letters on the footrests. Press the push-buttons on the footrest tubing and insert into the Fusion's frame. The footrests can be locked in two different positions (front, for transport chair use, and side, for rollator use). To rotate the footrests, press the push-button and rotate (**pic 1**).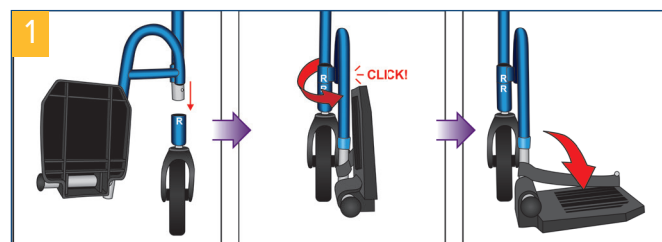

When not in use, the footrests can be conveniently stored by pushing the push-button and swinging them to the side of the frame where another push-button will hold them securely in place. This allows you to lift and store your Fusion safely. The footrests can also be removed when not in use.


TIP: the black cap on the footrest tubing can be removed and used to cover the open tubing where the footrests slot into.

Unfolding

Place the rollator on its wheels.

Now press both side rails of the seat section downward until they click into place (**pic 4**).

The rollator is unfolded correctly when the locking hook have clicked into place (**pic 5**).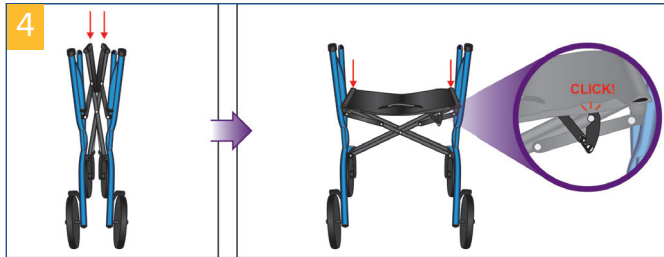

Assembly of accessories


Install the bag by sliding it onto the mounts starting from the back, and then pulling to the front.

Height adjustment of push handles

Insert push handles at least to the first cross hole into the frame tubing. Insert the fixing screw from the inside and tighten using the washer and star screw (**pic. 5-6**).


To change the height adjustment of the push handles, loosen the fixing screw by turning the star screw counter-clockwise. Then lock the handle at the desired height, as described above.

Hand tighten only.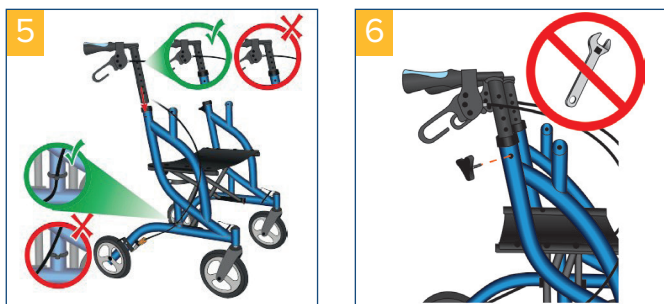

Adjusting the brakes

If braking is poor or too hard, the brake can be adjusted using the adjusting screw on the Bowden cable below to achieve the desired braking effect. To do this, press the adjusting sleeve in the direction of the wheel and then rotate the adjusting screw accordingly (**pic. 7**).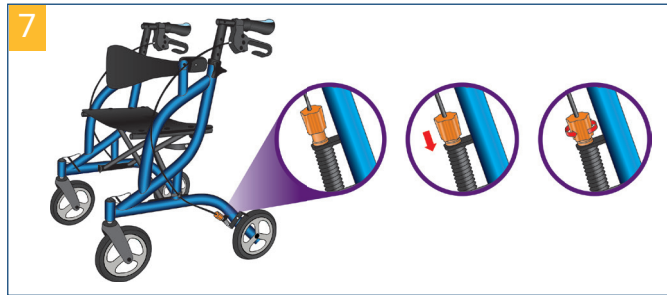


Clockwise: Brake looser
Anti-clockwise: Brake tighter

Important: Before using the rollator again, ensure the brakes are working properly. DO NOT use the rollator if it moves while the brakes are in the locked position. If you are unsure how to adjust your brakes, please contact your dealer.

Braking | Parking

To brake when walking with the rollator, pull both brake handles upwards with your hands (**pic. 8**). Always brake slowly while applying equal pressure with both hands.

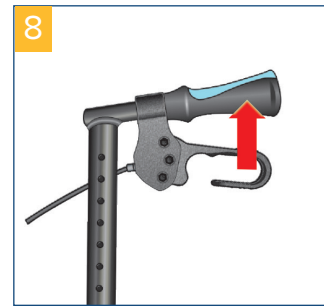To apply the brakes/park the rollator, press both brake handles downwards until they audibly click into place (**pic. 9+10**).


To release the brake, pull both brake handles upwards once again until they are released (brake handles are in the middle position) (**pic. 11**).

The braking effect can be impaired by wear and tear on the tyres. For this reason, regularly check the braking effect.

Use as a rollator


Flip the backrest away from the handlebars so that it is positioned towards the front wheels (**pic. 12**).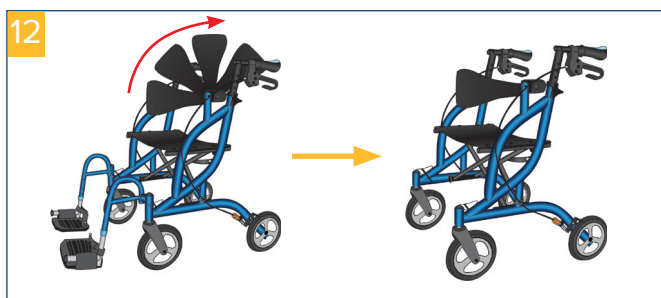

Sitting:

Place the rollator on an even and solid surface. Then apply the brakes. The rollator must be stable and may not slip. Only sit upright; do not lean backwards (pic. 13)!

Seated position permitted only as shown and against the direction of movement (**pic. 14**)!

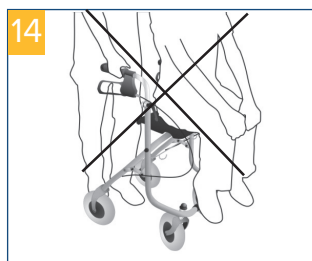Use as a transport chair

Flip the backrest toward the handlebars so that it is positioned to the back of the Fusion. Then, for each footrest, press on the push-buttons above the front wheel and rotate the footplate toward the front of the Fusion until it locks into place (**pic. 15**).


Before entering or exiting the seat, always ensure that the footrest plates are folded up and that the brakes are locked. Once the user is seated, deploy the footrest plates as illustrated.

Their height can be adjusted using the push-buttons on their tubing

Folding | Transporting | Stowing

To fold up the rollator, use the handle to pull the seat section upwards. (**pic. 16**).

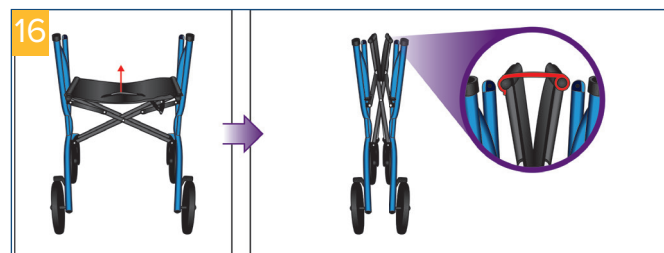Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60° C for long periods of time (for example, use in a sauna, etc.)!
- In the event of defects, contact your specialist dealer from whom you purchased this product!
- Check the condition of the tires regularly. They are “failsafe”, meaning that you do not have to check air pressure or fill them with air. Always check the tires for tears or other damage, however.

Technical data

Product	Airgo Fusion, blue
Item number	700-935INT
Recommended body size	1,55 - 1,85 cm
Turning radius	89 cm
Overall length x Overall width (Rollator)	73 x 67 cm
Overall length x Overall width (Transport Chair)	100 x 67 cm
Handle height	78,5 - 92,5 cm
Seat width x Seat depth	50 x 30 cm
Seat height	51,5 cm
Distance between Handgrips	53 cm
Folded size (LxWxH)	73 x 27 x 81 cm
Diameter wheels	ca. 202 mm / 8"
Weight w/o accessories	9,5 kg
Weight with accessories	9,8 kg
Weight capacity	136 kg
Weight capacity Bag	5 kg
Material wheels	PP + PVC
Material seat	Nylon
Material handgrips	PVC
Material frame	Aluminium, corrosion-protected, powdercoated

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Drive Medical GmbH & Co. KG