
KMINA

**USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE**

**KMINA - BED TRANSFER CHAIR
KMINA - SILLA TRANSFERENCIA CAMA
KMINA - BETT-TRANSFER-STUHL
KMINA - CHAISE DE TRANSFERT DE LIT
KMINA - SEDIA PER IL TRASFERIMENTO DEL LETTO**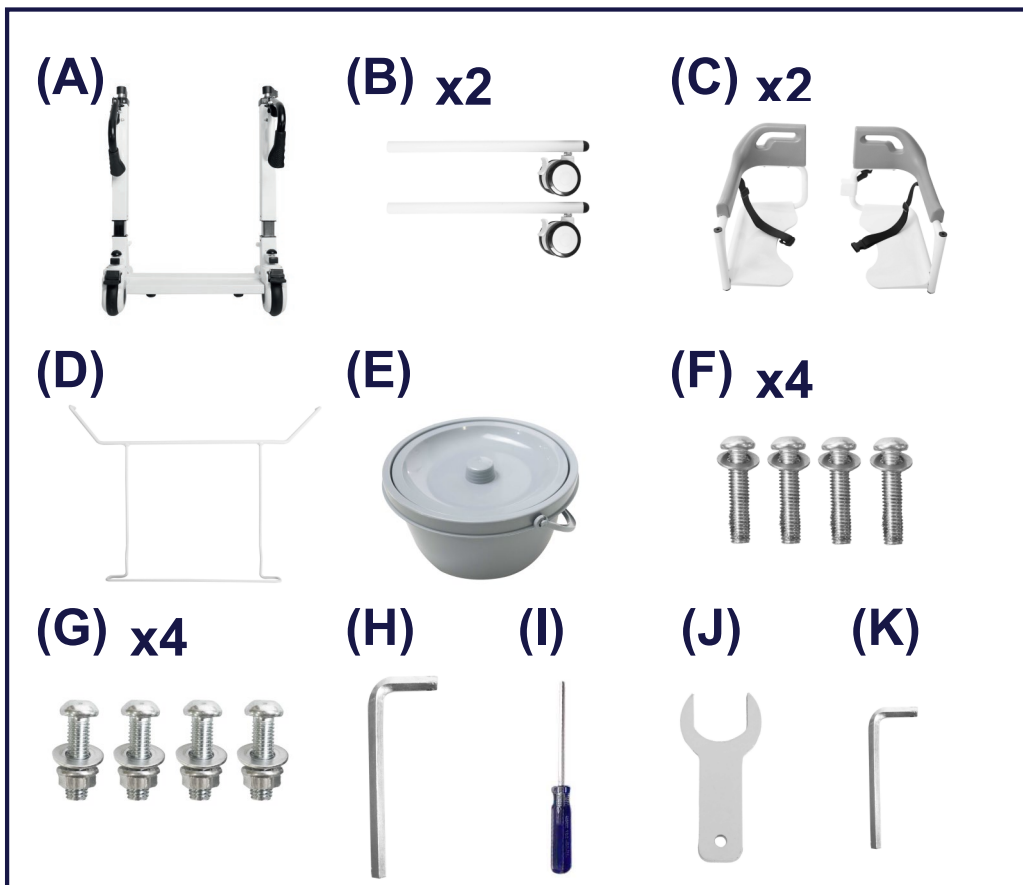

	ENGLISH	ESPAÑOL	DEUTSCH	FRANÇAIS	ITALIANO
A.	Main frame	Estructura central	Zentrale Struktur	Structure centrale	Struttura centrale
B.	Back legs	Patas traseras	Hinterbeine	Pattes arrière	Gambe posteriori
C.	Seat	Asiento	Sitz	Siège	Sedile
D.	Bucket rail	Rail del cubo	Nabenschiene	Rail du moyeu	Rotaia del mozzo
E.	Bucket	Cubo	Würfel	Cube	Cubo
F.	M8 washer and screw	Arandela y tornillo M8	Unterlegscheibe und Schraube M8	Rondelle et vis M8	Rondella e vite M8
G.	M10 washer, screw and nut	Arandela, tornillo y rosca M10	Unterlegscheibe, Schraube und Mutter M10	Rondelle, vis et écrou M10	Rondella, bullone e dado M10
H.	M6 Allen wrench	Llave Allen M6	M6 Inbusschlüssel	Clé Allen M6	Chiave a brugola M6
I.	Screwdriver	Destornillador	Schraubenzieher	tournevis	cacciavite
J.	Wrench	Llave inglesa	Schraubenschlüssel	Clé de serrage	Chiave inglese
K.	M3 Allen wrench	Llave Allen M3	M3 Inbusschlüssel	Clé Allen M3	Chiave a brugola M3

* Tools (J) and (K) are provided for support but are not required for assembly.

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1

EN 1. Support the main frame (A) on the floor so that the handles are resting on the floor for a more comfortable assembly. Position the back legs (B) by connecting the holes at the end with the holes next to the front legs and screw them together using the M10 bolt, washer and nut (G). Use the M6 Allen wrench (H) to tighten the 4 screws.

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EN 2. Position the central frame standing upright and fit each part of the seat armrest (C) into the hole in the central frame. Insert a screw and an M8 washer (F) into both the upper and lower holes and tighten with the help of the screwdriver (I).

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EN 3. Place the hub rail (D) on the supports of the central structure and then rest the hub (E) on it to complete the assembly.

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EN Place the transfer chair in the desired location and adjust the height of the chair until it touches the seating surface where the transfer is desired. Lock the brakes and unbuckle the seat belt and the backrest locking lever to unlock the seat so that the user can be dropped in the desired location.

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The **Bed Transfer Chair** is designed to facilitate the daily life of caregivers of patients using wheelchairs, helping them to transport and / or transfer users from one place to another in an easy and comfortable way. It is height adjustable, so it can be adapted to seats of different levels.

MATERIALS

Steel structure. PP, PVC and PA plastics.

WARNINGS

■ Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.

Pre-use checks:

- Read the instructions before installation.
- Confirm that all components and parts are properly assembled and screwed together before use.
- Be sure to fasten the safety buckle and seat belt before using the transfer chair.

During use:

- Be sure to lock the wheels when getting in or out of the transfer chair.
- If someone sitting in the transfer chair wants to reach for an object, they should not lean too far in case the chair tips over due to weightlessness.
- When adjusting the seat height, the front wheels must be locked.
- The wheelchair should travel on the smooth floor and avoid bumps or obstacles in case of tipping over.
- When traveling downhill on a slope of more than 5 degrees, the chair should face upwards (user faces upwards). The operator holds the plastic backrest to move the chair downhill in reverse. If the chair faces downhill when going downhill, the user may lean forward or even fall.
- Do not exceed the maximum user weight: 100 kg.
- Be sure to lock all 4 wheels when the chair remains on slopes.

CLEANING AND MAINTENANCE

- Do not scratch or scrape the surface of the product with hard objects to avoid damaging the material.
- For daily cleaning and maintenance, wipe the product with a damp cloth with neutral synthetic detergent and then wipe off the dirt with a dry cloth.
- Keep away from strong acids and strong alkalis to avoid corrosion.
- Periodically check for loose nuts or bolts or damaged or cracked parts. If necessary, replace damaged parts.

WARRANTY

KMINA warrants this device to be free from defects. The warranty will remain in effect for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear requiring periodic replacement. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorized dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not liable for accidental damage or damage to property.

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